

seca 402

seca 403

Scale shown as example
For compatibility information see section 14.

CE


This section contains four diagrams illustrating safety and assembly instructions:

- Top Left:** A diagram showing the correct way to lift the cart by the handles, marked with a sad face (frowning mouth) to indicate it is incorrect or unsafe.
- Top Right:** A diagram showing the correct way to lift the cart by the handles, marked with a happy face (smiling mouth) to indicate it is the correct and safe method.
- Bottom Left:** A diagram showing the assembly instructions booklet for the Seca 402/403 Baby scale carts, with an arrow pointing to a person reading it.
- Bottom Right:** A diagram showing two people holding hands, indicating that two people are required to move the cart safely.


Technical drawing of the three-wheeled mobile cart showing three views: front, side, and top.

- Front View:** Shows a rectangular frame with two horizontal shelves. The width is labeled as 79,5 cm.
- Side View:** Shows the profile of the cart. The height is labeled as 80 cm. The depth is labeled as 50 cm.
- Top View:** Shows the cart from above. It indicates the weight capacity for each shelf:
 - Top shelf: max 12 kg
 - Bottom shelf: max 6 kg
 - Total capacity: max 45 kg


4. SECA 402 SCOPE OF DELIVERY

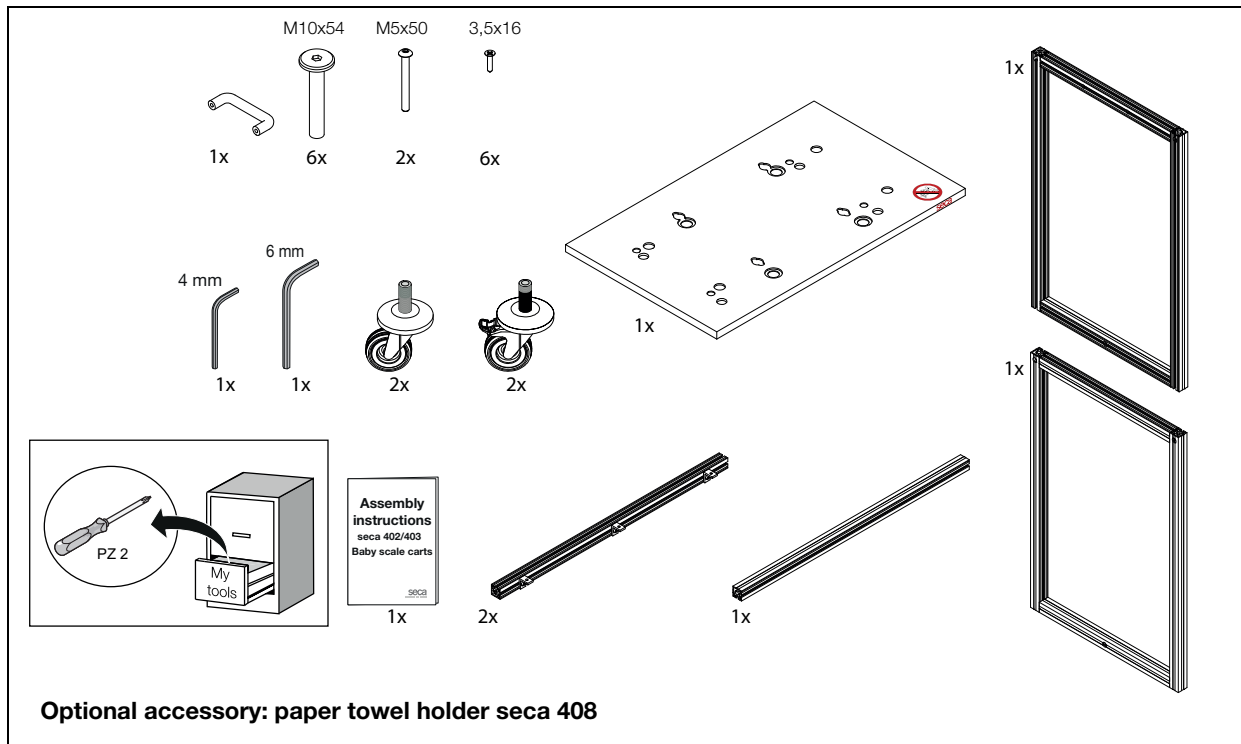


5. SECA 403 SCOPE OF DELIVERY

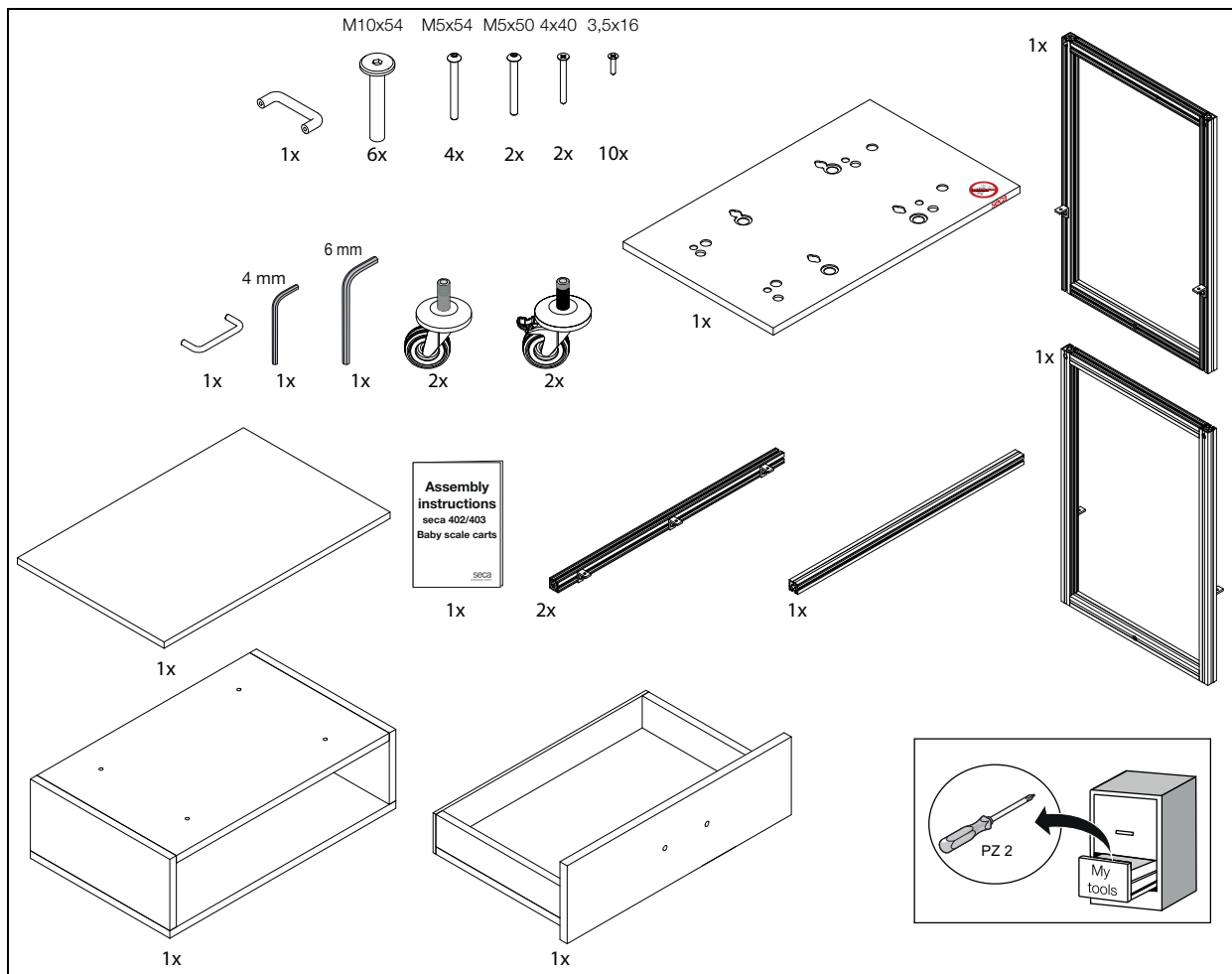









6. ASEMBBLING THE FRAME

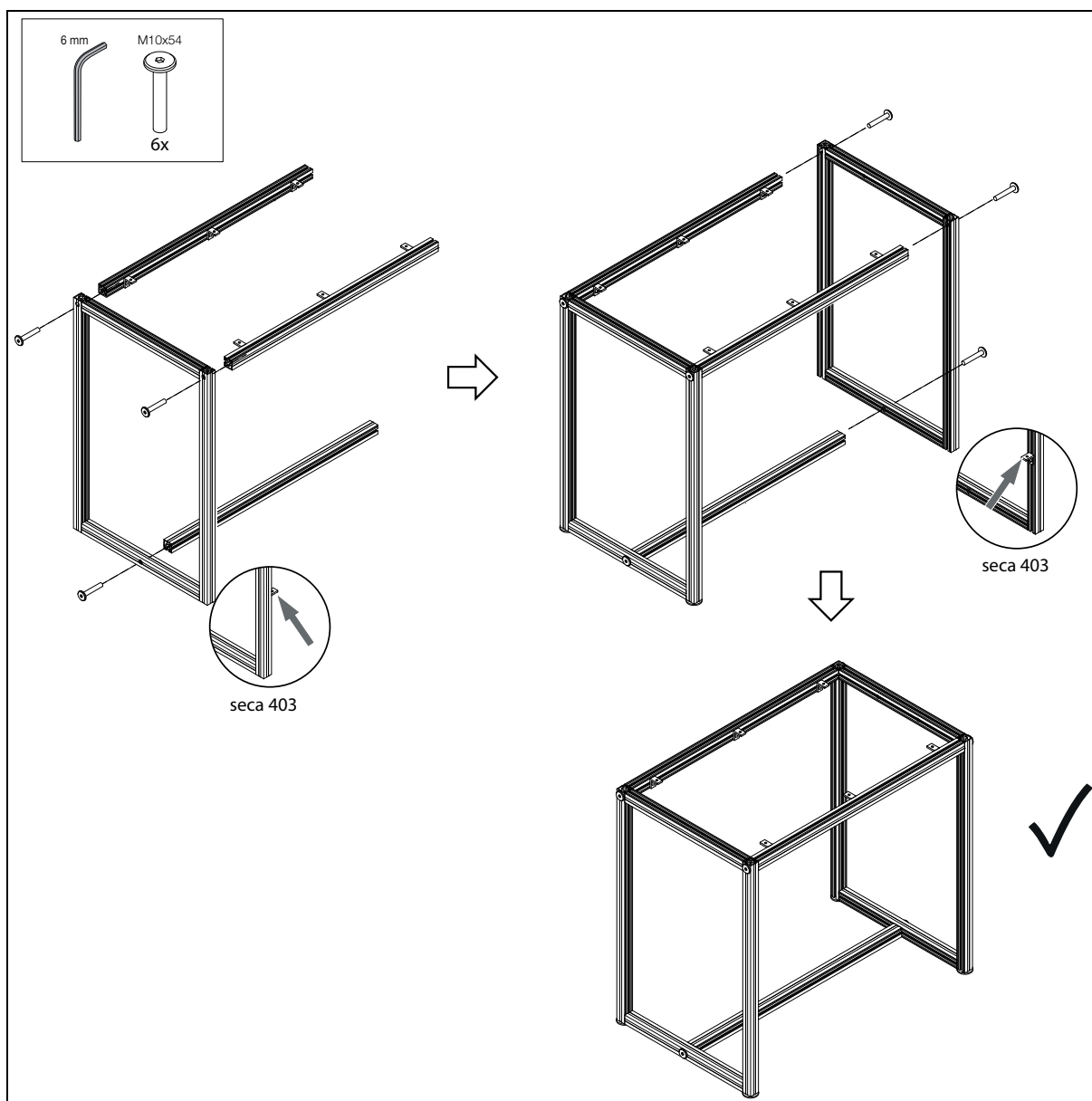

7. MOUNTING THE HANDLE

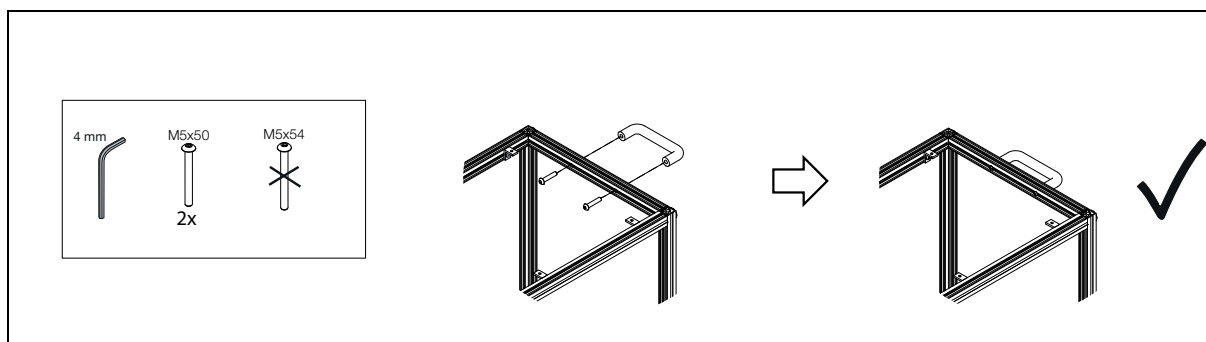


8. MOUNTING THE CASTORS (SECA 402 SHOWN)

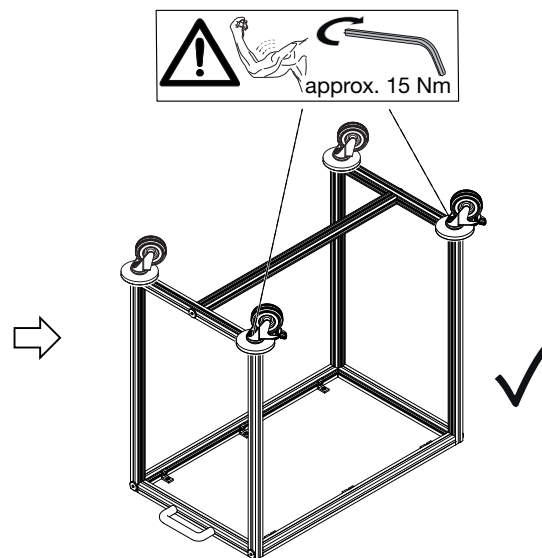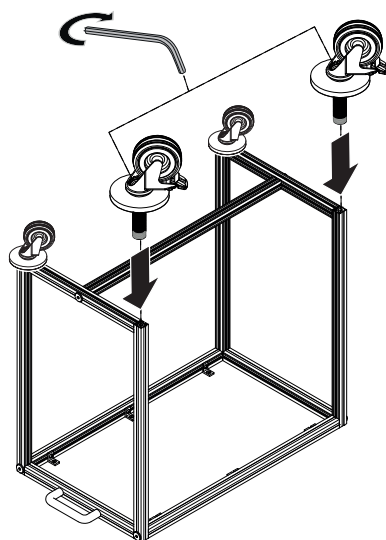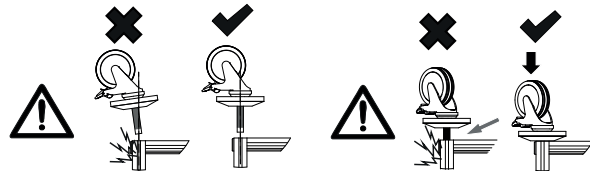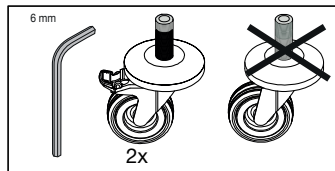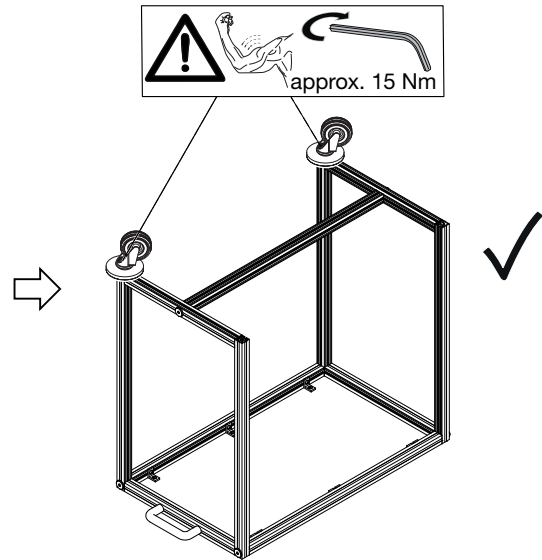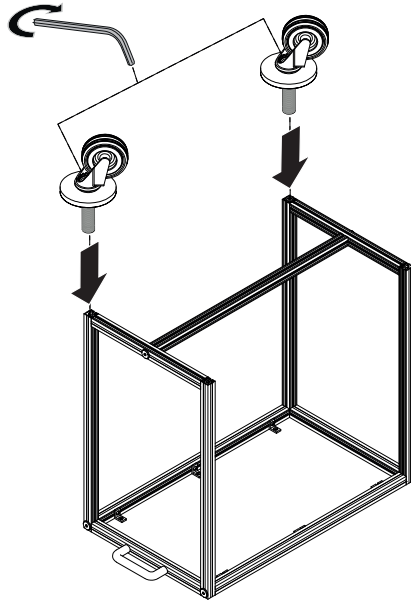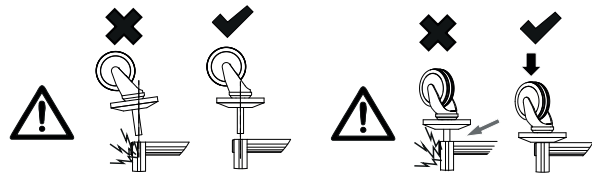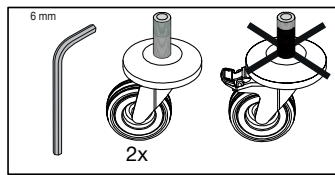


Check and retighten **all** castors at least once every year!


9. MOUNTING THE TOP SHELF

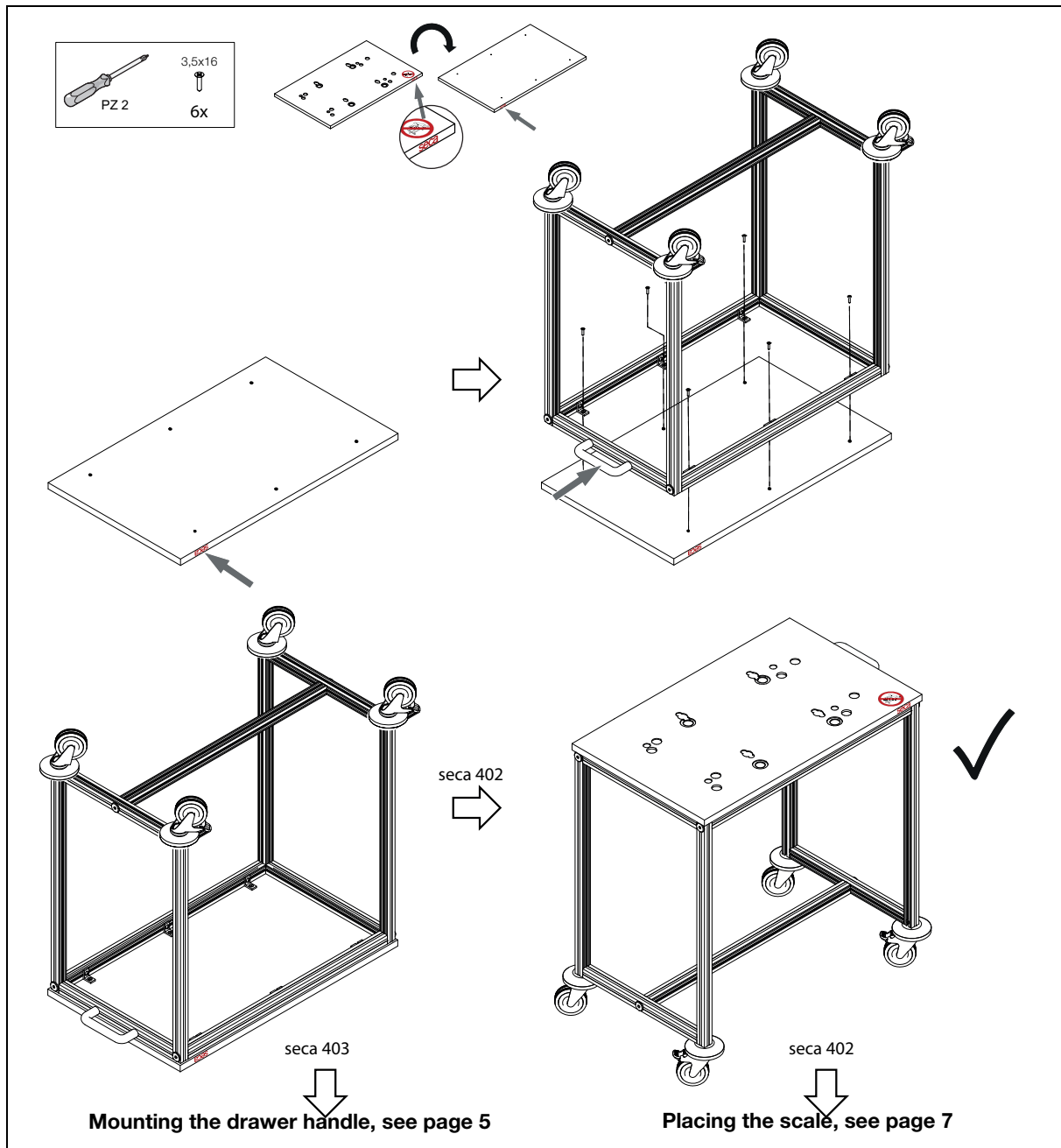


10. MOUNTING THE DRAWER HANDLE (SECA 403 ONLY)

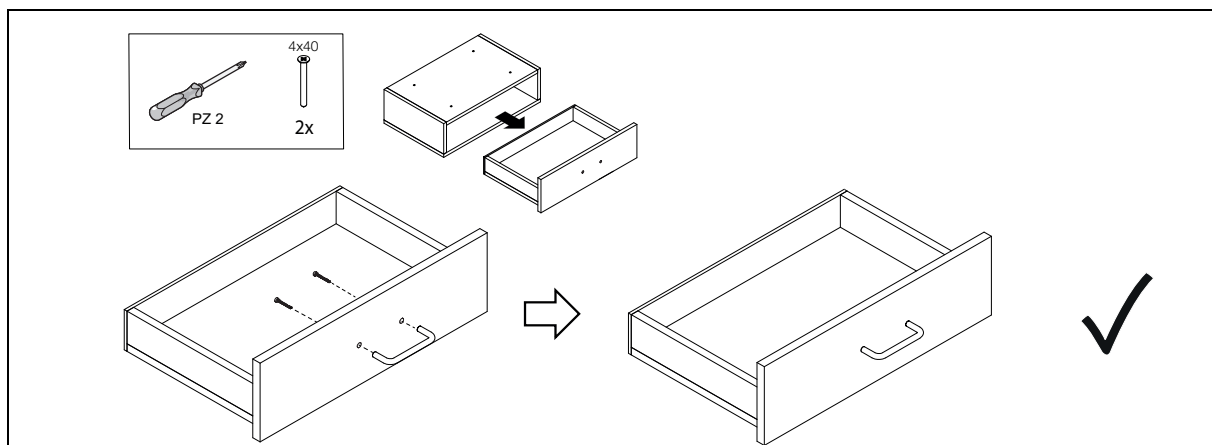


11. MOUNTING THE DRAWER (SECA 403 ONLY)

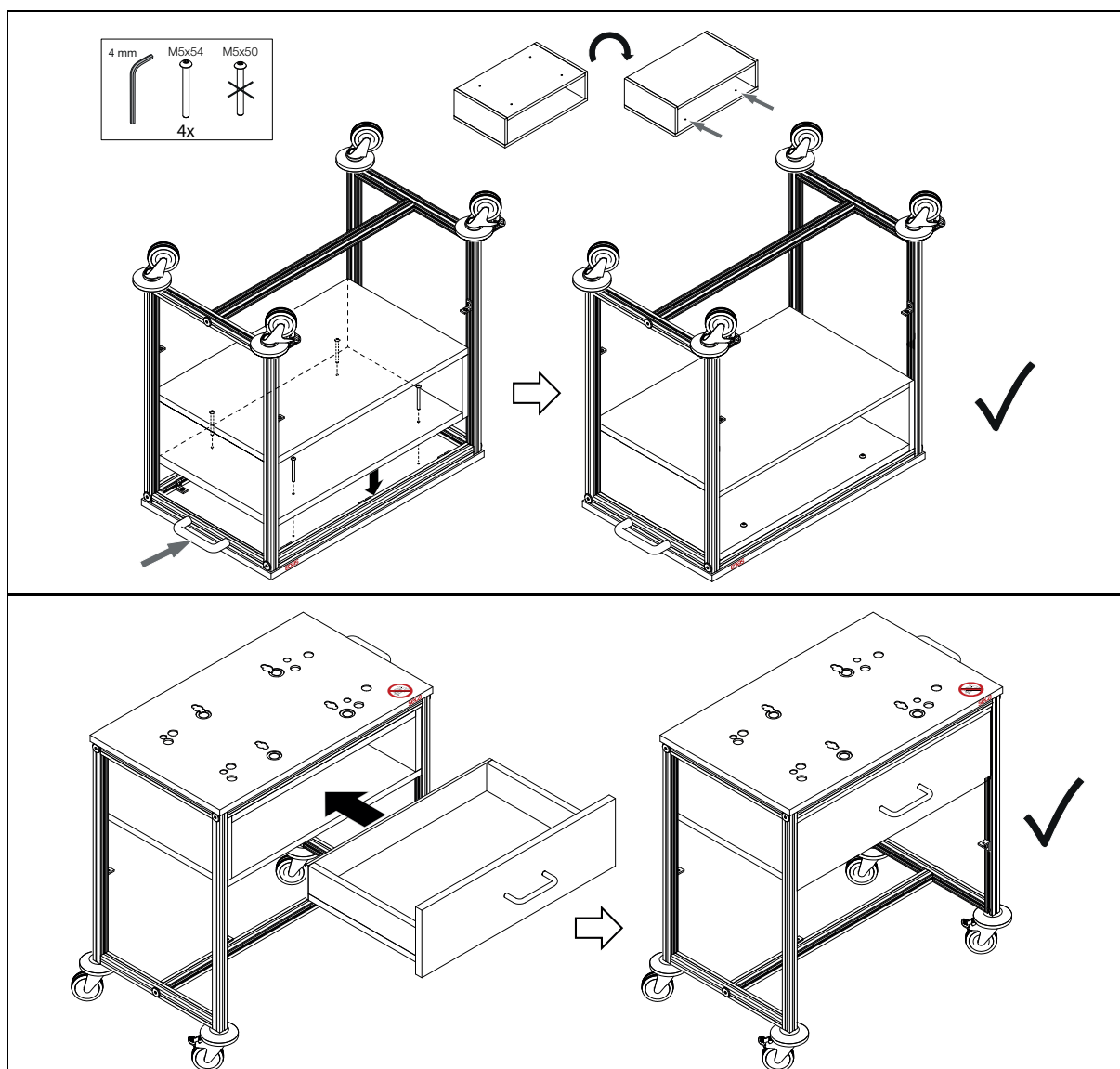







12. MOUNTING THE SHELF (SECA 403 ONLY)

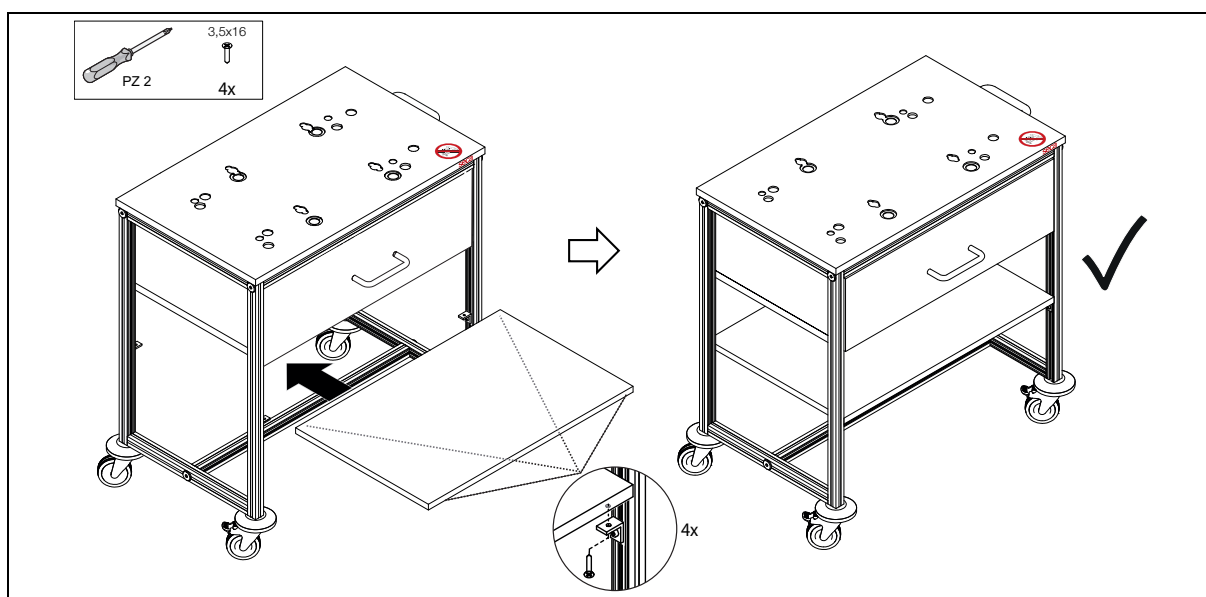


13. PLACING THE SCALE (SECA 402 SHOWN)

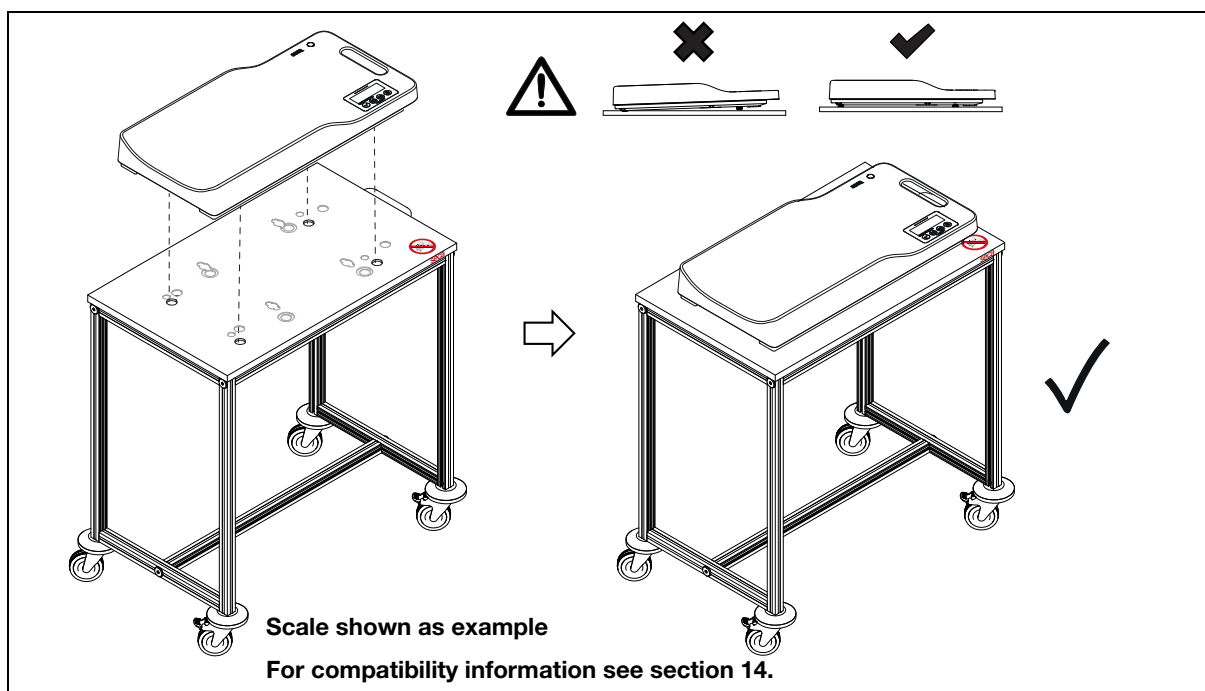

14. COMPATIBLE SCALES (SECA SCALES ONLY)

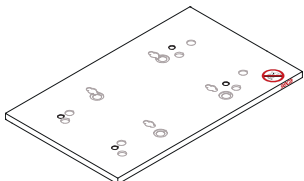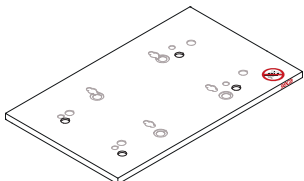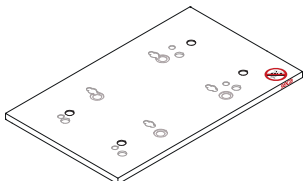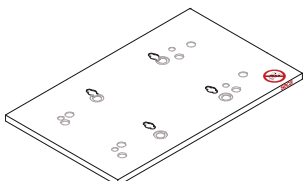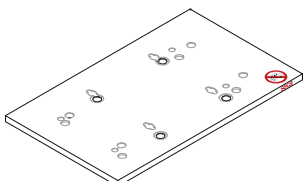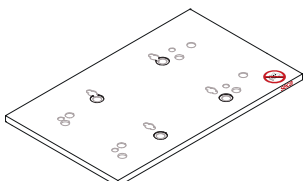Patterns for seca scales		
seca 334 / seca 335 	seca 336 / seca 336i / seca 333i 	seca 378 / seca 376 / seca 374 
seca 354 / seca 382 / seca 383 / seca 833 / seca 834 / seca 835 / seca 385 / seca 384 	seca 727 / seca 728 	seca 757 / seca 717 

15. PRIOR TO WEIGHING

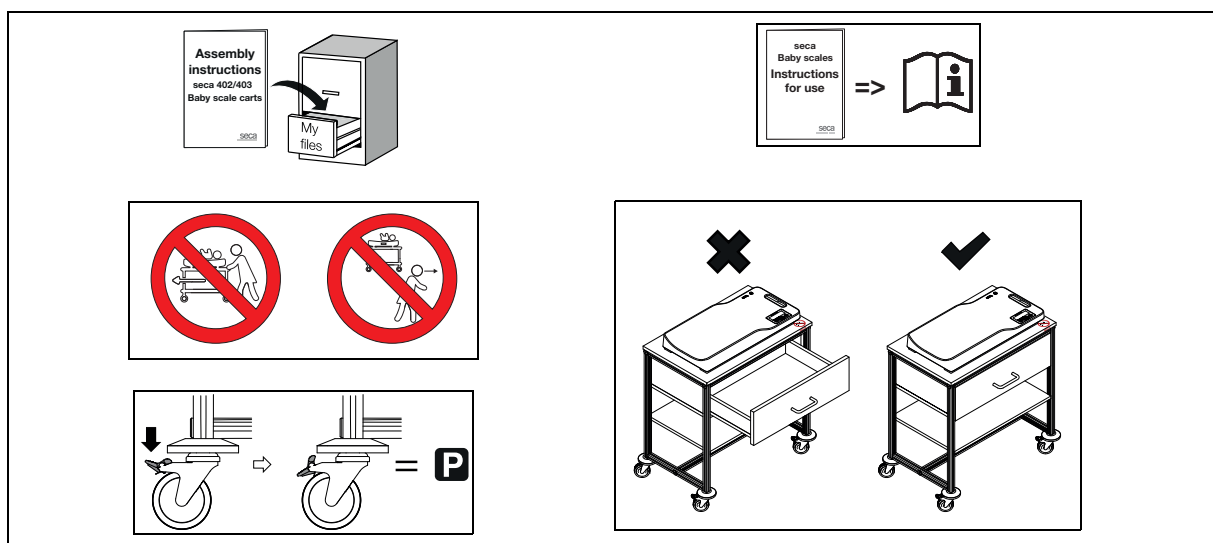

16. HYGIENIC TREATMENT

Cleaning	Disinfecting
NOTICE: ▶ Do not use aggressive or abrasive cleaning agents. ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit). 1. Moisten a soft cloth with a soap solution. 2. Wipe over all surfaces. 3. Rinse thoroughly with hot water. 4. Dry surfaces with a soft cloth.	NOTICE: ▶ Follow the instructions on the disinfectant. ▶ Test disinfectant beforehand on a small, non-visible area. 1. Moisten a soft cloth with disinfectant. 2. Wipe over all surfaces. 3. Allow disinfectant to act for the time indicated in the disinfectant's instructions. 4. Rinse thoroughly with water. 5. Dry surfaces with a soft cloth.

Medical Measuring
Systems and Scales
since 1840

seca gmbh & co. kg
Hammer Steindamm 3-25
22089 Hamburg
Germany

T +49 40 20 00 00 0
F +49 40 20 00 00 50
E info@seca.com

seca operates worldwide
with headquarters in
Germany and branches in:

- seca france
- seca united kingdom
- seca north america
- seca schweiz
- seca zhong guo
- seca nihon
- seca mexico

- seca austria
- seca polska
- seca middle east
- seca brasil
- seca suomi
- seca américa latina
- seca asia pacific
- seca danmark
- seca benelux
- seca lietuva

and with exclusive
partners in more than
110 countries.

All contact data at
seca.com